3Delta Denture Base pink-opaque 1000g

REF 1606020

3Delta Denture Base pink-transparent 1000g

REF 1606120

3Delta Denture Bond 10 ml

REF 1606950

3Delta Denture Fix 8g

REF 1606910

de	Gebrauchsanweisung
en	Instructions for Use
fr	Mode d’emploi
es	Instrucciones de uso
it	Istruzioni per l’uso
pt	Instruções de utilização

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INSTRUCTIONS FOR USE en

Description:

3Delta Denture Base is a light-curing, acrylate-based material for additive production of denture bases. The material has been optimized for Rapidshape® DLP® systems in the D series with a wavelength of 385 nm for layer thickness of 100 µm. Denture teeth are bonded to the denture base using **3Delta Denture Bond** and **3Delta Denture Fix**. The material meets the performance characteristics of DIN EN ISO 20795-1 Type 4.

Composition:
Preparation of acrylates and initiators.

Indication for use:

- 3Delta Denture Base** – Light curing resin for additive manufacturing of denture bases.
- 3Delta Denture Bond** – Light curing bonding agent for denture teeth and denture bases.
- 3Delta Denture Fix** – Dual curing cementation composite for denture teeth and denture bases.

Contraindications:
Patient and user: Products should not be used in patients or by users who are known to be allergic to any of the ingredients. Otherwise the patient group is not restricted.

Adverse reactions:
Patient and user: Product can cause allergic reactions.
User: Observe warnings.

Interactions:
Interactions are not known.

Processing instructions:
BUILD PROCESS
➤ Prepare Data: Align parts in labial direction towards the build platform. The type of support structures can be found in the instructions for use of the system supplier.
➤ Define variable process parameters: The selection of suitable process parameters is carried out according to the specifications of the printer system supplier. Care must be taken to select the correct material-specific parameter set (product, layer thickness, exposure time).
➤ Stir the material thoroughly and bubble-free and pour it into the material reservoir of the printer: Let the material rest for 10 min in the material reservoir.
➤ Start build process and print part

CLEANING AND DRYING

➤ Clean part: The objects are cleaned under flowing conditions with isopropanol for max. 5 min. If necessary, carefully clean the objects with a brush and a spray bottle filled with isopropanol.
➤ Dry part: Blow off part with compressed air and allow to dry for 30 min at room temperature or in drying cabinet heated to 37° C.
➤ Remove support structure

POST CURING

➤ For post-exposure, the components must be dry and cooled down to room temperature.
➤ Post-curing is done with

- light units RS Cure (Rapidshape) with selection of the corresponding parameter set of the material. [Exposure time: 10 min / vacuum 50 mbar / without nitrogen / heater off / power 100 % / wave length 375+415 nm]

or

- light unit Otofash G171, (Nk Optik) with 2000 flashes each for top and bottom of the denture (Σ 4000 flashes).

➤ The device used must be maintained and calibrated according to the manufacturer's specifications to ensure mechanical properties, color and biocompatibility.

➤ POST PROCESS FINISHED

Denture finishing:

- Sandblast the teeth (circular/basal) and the printed base (alveolar) on the bonding surfaces with 110µm Al₂O₃ at 2 bar.
- Steam off and dry the teeth and base.
- Place a few drops of **3Delta Denture Bond** on a mixing plate and wet the teeth and base at the sand-blasted areas. When applying the bonding agent, make sure to apply it as thin as possible in order to

- avoid ponding. Cure in the RS Cure polymerization unit (Rapidshape), presetting "**3Delta Denture Bond**" (3.5 min / 100% / 375+415 nm) or Otofash G171, (NK-Optik) with 1000 flashes
- Place the Automix cannula on the **3Delta Denture Fix** cartridge. Squeeze out an amount approximately the size of a lentil and discard.
- Carefully apply **3Delta Denture Fix** in the alveoli of the denture base and place corresponding teeth. Make sure to use the material as sparingly as possible to avoid excessive squeezing out between the base and the teeth. Remove any excess immediately with an instrument. The working time of **3Delta Denture Fix** is 5 min.
- Self-curing is completed after 15 min. Do not use pressure curing units. Afterwards, post-cure the denture for 3.5 min in the RS Cure (Rapidshape). Preset "**3Delta Denture Fix**" (3.5 min / 100% / 375+415 nm) or Otofash G171, (NK-Optik) with 1000 flashes.
- The final finishing and polishing is carried out according to the usual dental techniques.
- The fit of the printed restorations must be checked and assessed by both the dental technician (on the model) and the dentist (in the patient's mouth).

Warnings for the user:

Any deviation from the described processing procedure may affect the biocompatibility, functionality as well as the safety of the final product.
3Delta Denture Base, 3Delta Denture Fix, 3Delta Denture Bond: Please always observe the respective valid safety data sheets. May cause an allergic skin reaction. Avoid contact with uncured material. Wear protective gloves / protective clothing / eye protection. If on skin: Wash gently with plenty of soap and water. If skin irritation or rash occurs: get medical advice/attention. If in eyes: Rinse cautiously with plenty of water for several minutes. Remove contact lenses if possible. Continue rinsing. Avoid inhalation of dust/vapours/ spray. Switch on extraction system. The products are hazardous to the environment. Collect spillage

Additionally for **3Delta Denture Bond:** Highly flammable liquid and vapours. Causes skin irritation. Causes serious eye damage. May cause respiratory irritation. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

Always observe the relevant country-specific health and safety regulations and legal requirements when handling hazardous substances.

Storage instructions / shelf life:
Protect from exposure to light. May polymerize spontaneously. Store between 4°C and 25°C (39°F – 77°F). Keep container tightly closed. Do not use after the expiration date. See imprint/label for expiration date.

Disposal instructions:
Dispose of contents/container in accordance with local regulations.

The product was developed for use in dentistry and must be used according to the instructions for use. Serious incidents must be reported to DeltaMed GmbH and the responsible authority.

LOT	Batch code		Manufacturer
REF	Catalogue number		Keep away from sunlight
	Use-by date		Exclamation mark
	Consult instructions for use		Environmental hazard
	Temperature limit		Caustic effect
MD	Medical Device		Flame
	Stir before use		

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