3Delta Guide S

Gebrauchsinformation

Deutsch

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English

Description:

3Delta Guide S is a light-curing material for the production of precise dental drill guides. It is optimized for DLP®-Systems with a wavelength of 385 nm. The material has been optimized for Rapidshape DLP®-systems with a wavelength of 385 nm for layer thicknesses of 100 µm.

Composition:

Composition of acrylates and initiators.

Indication:

- 3D-Printig of dental drill guides.

Contraindication:

Patient and user: **3Delta Guide S** must not be used if an allergic reaction to any of the ingredients is known. Otherwise patient group not restricted.

Side-effects:

Patient and user: Product may cause allergic reaction.

User: Respect warning notes. Unpolymerized material causes skin and serious eye irritation.

Interactions:

Interactions are not known.

Warning notes:

Please always pay attention to the current safety data sheet!

3Delta Guide S contains acrylates and phosphine oxides.

Avoid breathing vapours, use extraction unit. Avoid contact respectively wear protective gloves, protective clothing, eye protection. If on skin: wash with plenty of water. If skin irritation occurs: Get medical advice. Uncured material is harmful to environment. Avoid release to the environment

Advice on handling isopropanol during drying and cleaning process: Isopropanol is a hazardous good. You must use a suitable extraction system when working with isopropanol in order to prevent the creation of an explosive atmosphere. Otherwise, explosion-proof devices must be utilized. Consequently observe the valid safety data sheets as well as country-specific legal regulations when handling hazardous substances.

Storage instructions / Shelf Life:

Protect uncured material from light. May polymerize spontaneously. Do not use after the expiration date Store between 4°C and 25°C.

Disposal instructions:

Dispose of contents/container in accordance with local regulations.

Delivery units:

Art.-Nr. 1605230

3Delta Guide S 500 g

Art.-Nr. 1605240

3Delta Guide S 1000 g

Important product information

Federal (U.S.) law restricts the finished device to sale by or on the order of a dentist.

The product was developed for use in the dentistry and must be used according to the instructions for use. The Manufacturer does not accept any liability for damage caused by improper use of system and material. Furthermore, the user must independently check the material for its suitability and applicability for the intended purposes before using it, especially if he diverts from the process described in the instructions for use, or if his process is not described in the instructions for use. Serious incidents must be reported to DeltaMed GmbH and the responsible authority.

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
Application: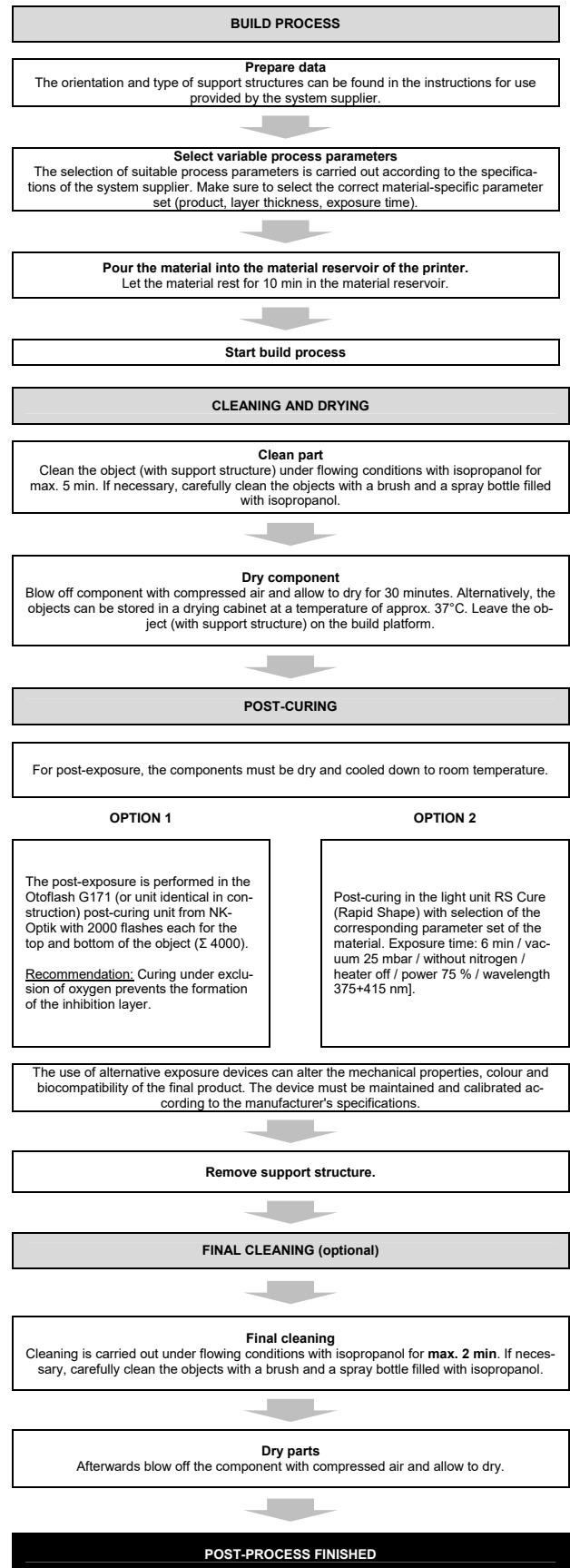

Further processing information:

Parts made of **3Delta Guide S** are finished according to the usual dental technical aspects.

Information on optional sterilisation:

Steam sterilizable in autoclave for 4 min. at 132°C and subsequent drying for 30 minutes.